BiTech Bipolare Scheren

BiTech Bipolar Scissors

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BiTech bipolar scissors enable cutting and dissection with simultaneous bipolar coagulation. They can be used for a large number of applications in open surgery. The patented construction principle ensures that the scissors are both easy to use and to repair. The electrical contacts for both scissor blades are conducted via one section of the handle, i.e. the connection cable only needs to be attached to one ring, which significantly reduces interference during handling.

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BiTech Bipolare Scissors with standard blade

The standard version of the BiTech scissors are characterized by their strong blade and large contact surface for use i.e. in general surgery and gynecology.

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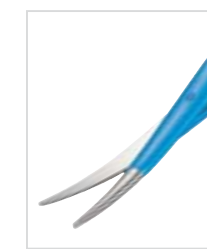
BiTech Bipolare Scissors with delicate blade

The more delicate blades of these models with smaller contact surface are suitable i.e. for use in thoracic surgery, gynecology and ENT.

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BiTech Micro

The BiTech Micro scissors are available in 13, 14 and 17,5 cm lengths and are characterised by their particularly slim and thin scissor blades. This is a major advantage in applications requiring high precision surgery e.g. plastic and reconstructive surgery.

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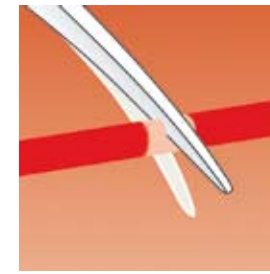
BiTech Supercut

The razor-sharp blade of the BiTech Supercut scissors enables particularly precise and fine cutting combined with simultaneous or alternate coagulation of tissue.


Bissinger is the first manufacturer to integrate the well established SUPERCUT scissor blade into bipolar scissors


Koagulieren während des Schneidens
Coagulation during cutting


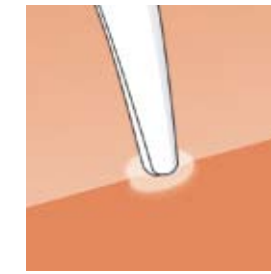
Koagulieren vor dem Schneiden
Coagulation prior to cutting

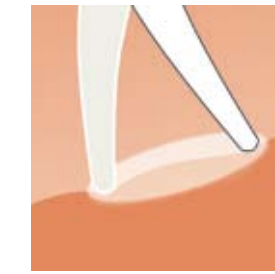Punktförmige Koagulation
Pinpoint coagulation

Flächige Koagulation
Zone coagulation

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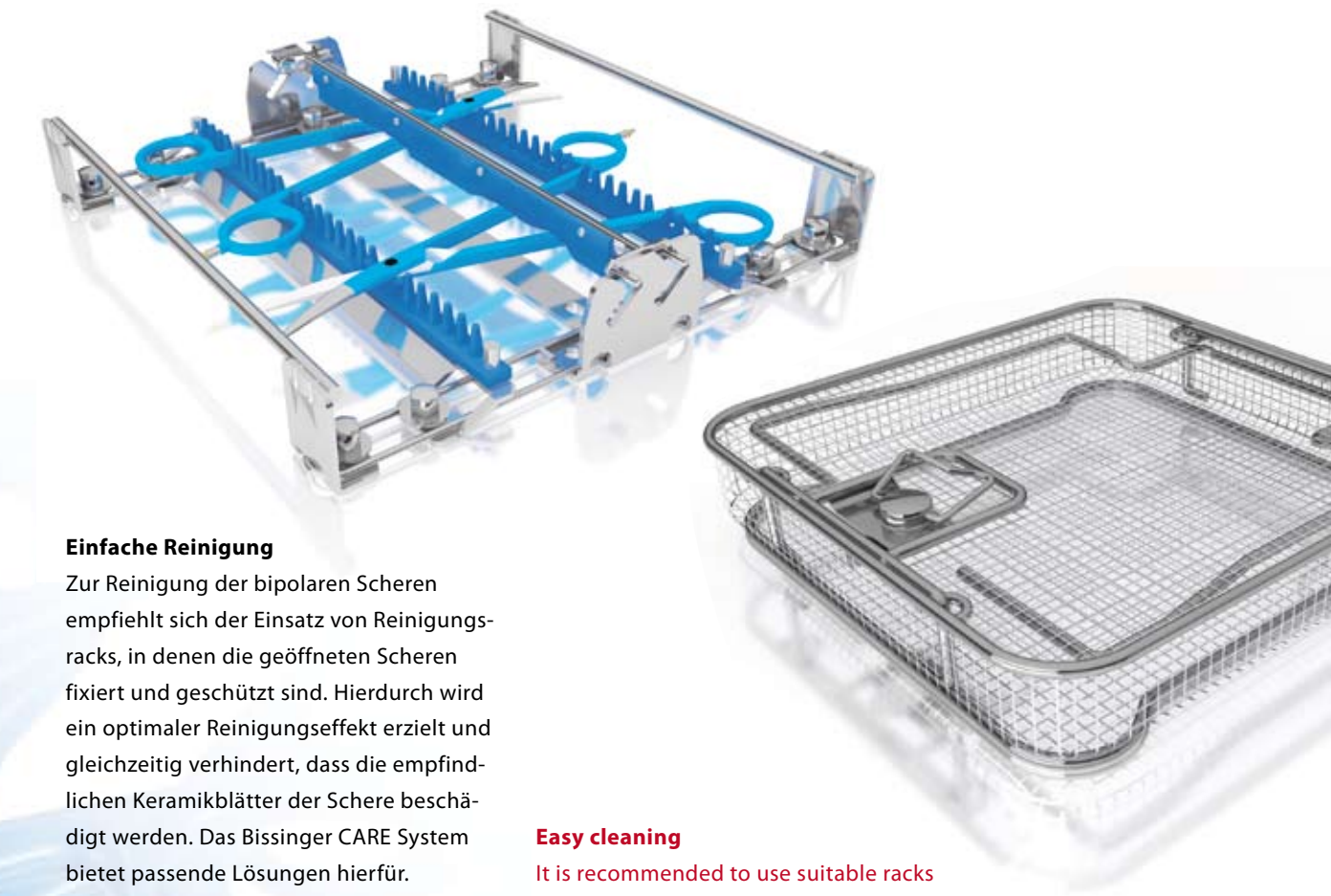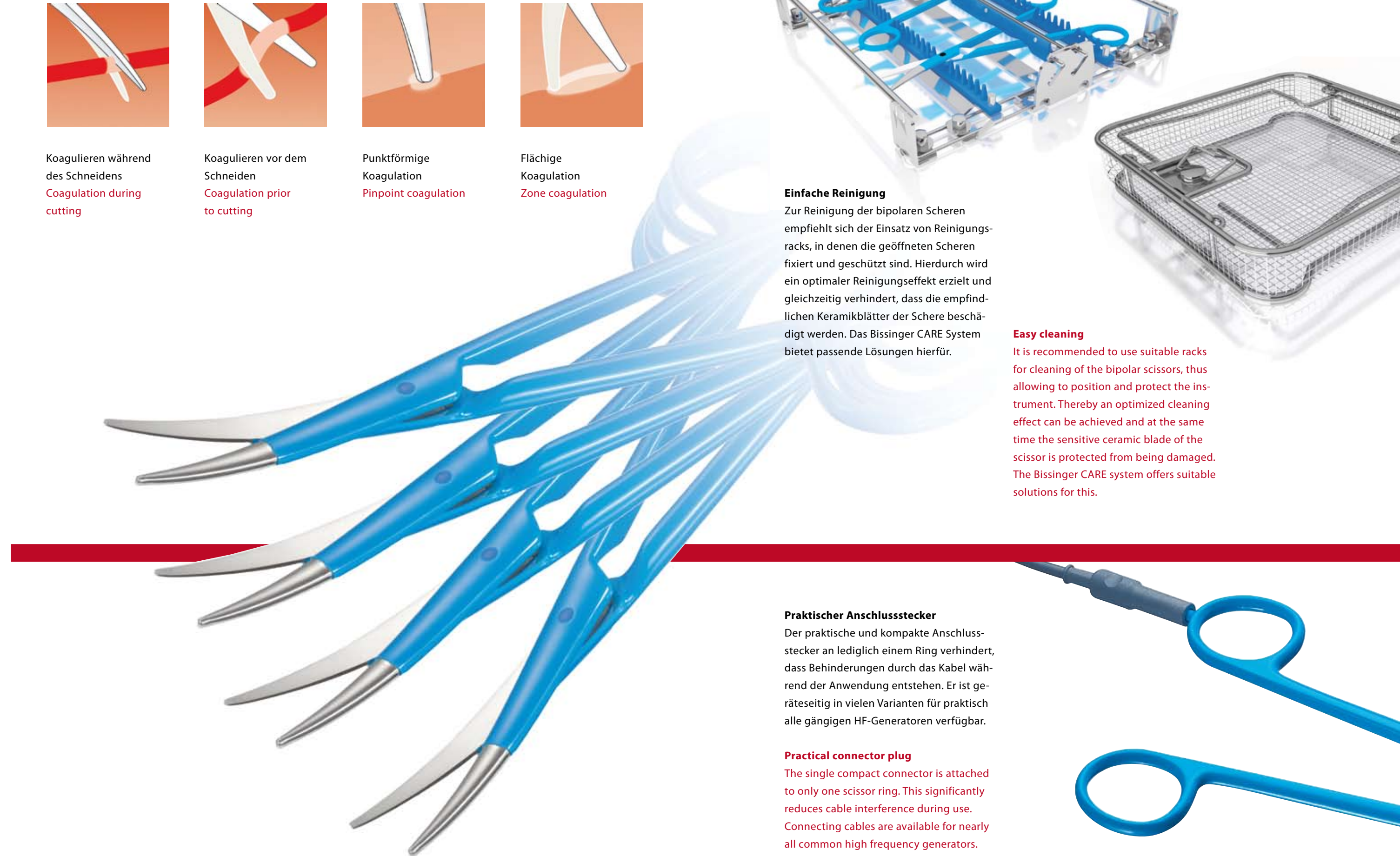Easy cleaning

It is recommended to use suitable racks for cleaning of the bipolar scissors, thus allowing to position and protect the instrument. Thereby an optimized cleaning effect can be achieved and at the same time the sensitive ceramic blade of the scissor is protected from being damaged. The Bissinger CARE system offers suitable solutions for this.

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Practical connector plug

The single compact connector is attached to only one scissor ring. This significantly reduces cable interference during use. Connecting cables are available for nearly all common high frequency generators.

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Cables are available with connectors for most electrosurgical generators.

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Our Products and Services

- Surgical instruments
- Bipolar and monopolar instruments
- Accessories for Electrosurgery
- Development of instruments to customer specifications
- Service and repair

Bissinger – Focus on better Solutions

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Medizintechnik GmbH